

Clinically, according to the different lesions and surgical sites, stoma can be divided into:

1. Colostomy

With 1 - 1.5 cm higher than abdominal wall and a diameter about 3 - 5 cm, the colostomy is usually located at left hypogastrum, and is a permanent stoma for colon or sigmoid. The feces excreted from here is usually shaped.

2. Ileostomy

With 1.5 - 2.5 cm higher than abdominal wall and a diameter about 2 - 2.5 cm, the ileostomy is usually located at right hypogastrum, and is the opening of distal ileum in abdominal wall. The feces excreted from here is usually liquid and rich in digestive enzymes causing strong thrill to the surrounding skin.

3. Temporary Stoma

The stoma is located in transverse colon segment, that is, the epigastrum, and is double-barrel or loop stoma with large appearance. The proximal stoma excretes loose stool, while the distal stoma excretes only a small amount of intestinal mucus. Temporary stoma plays the role of shunt and decompression, which can be closed when the diseases of lower segment of intestinal heal.

4. Urostomy

The urostomy is usually located at right hypogastrum, but sometime can be determined by the doctor according to the actual situation of surgery. Use a segment of free ileum to replace bladder. With 2 - 3 cm higher than abdominal wall and a diameter about 2 - 2.5 cm, the stoma is located at abdominal wall. After surgery, urine is discharged through this stoma.

临床上根据病变和手术部位的不同可分为：

1. 结肠造口

一般位于左下腹，为降结肠或乙状结肠永久性造口，突出腹壁约1-1.5cm，直径约3-5cm，排出的粪便通常是成型的。

2. 回肠造口





一般位于右下腹，是回肠末端在腹壁的开口，突出腹壁约1.5-2.5cm，直径约2-2.5cm，排泄物为流质状，富含消化酶，对造口周围皮肤刺激性强。

3. 临时性造口

位于横结肠段，即上腹部，为双腔或襻式，故外观较大。近端开口排出稀便，远端开口仅排出少量肠黏液。临时性造口起分流、减压作用，当下段肠道疾病愈合之后，可回纳。

4. 尿路造口

一般尿路造口位于右下腹，但有时医生会根据手术实际情况确定部位。它是游离一段回肠替代膀胱，在腹壁上开口，其直径约2-2.5cm，突出腹壁约2-3cm。手术后尿液通过此开口不断流出。

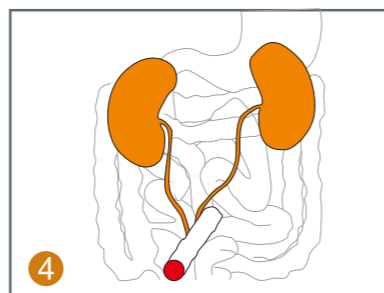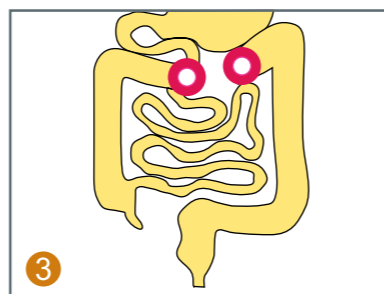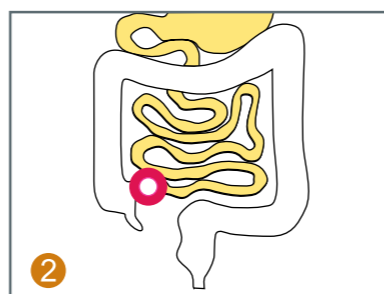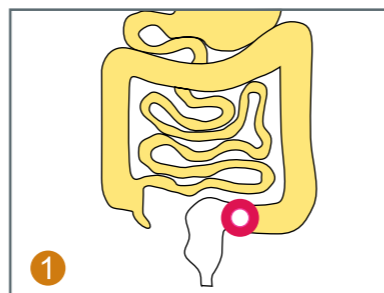


Advantage 1

产品优势 1

Special viscose formula, safe and reliable
特殊粘胶配方，安全可靠

Due to the special adhesive formulation, it is characterized by safe and reliable special formulation of hydrocolloid skin barrier, has good initial and lasting adhesive ability, and is friendly to skin as it contains no sensitization and stimulation, but resistant to acid, alkali (PH3-12), and various digestive enzymes. It can be pasted for 5-7 days if used correctly.

特殊配方的水胶体底盘，初粘性、持粘性好。对皮肤友好，无致敏，无刺激，耐酸碱(PH3-12)，耐各种消化酶。使用正确的情况下可粘贴5-7天。

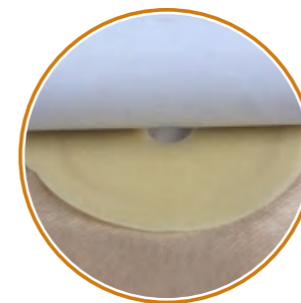


Advantage 2

产品优势 2

Odor obstruction
阻隔异味

It can perfectly obstruct the odor and has special fragrance with high tear strength in longitudinal and transverse direction.

阻隔异味性极佳，并有特殊的香味，纵横向撕裂强度高。

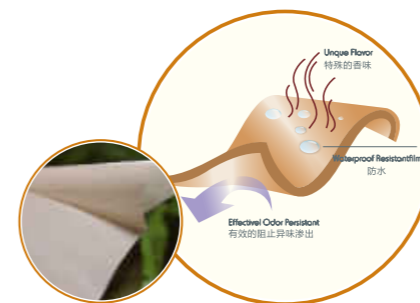

Advantage 3

产品优势 3

High efficiency activated carbon filter
高效活性炭过滤片

The high efficiency filter made of activated carbon in special process can effectively prevent odor escaping from pouches.

高效及特殊工艺的活性炭片，有效防止袋内异味的逸出。



Advantage 4

产品优势 4

Soft and compact two-piece ring
柔软密合的二件式开环

The soft and compact two-piece ring can embedded firmly and tightly but easy to open.

柔软又密合的二件式卡环，扣合牢固、严密，开启方便。



Advantage 5 Connection type (patented design)
产品优势 5 专利设计连接方式

The ring (patented design) connected barrier with film to keep the ring at the same level and not easy to fall off.

专利设计的卡环和底盘膜连接，使卡环保持在同一水平，不易脱落。



Advantage 7 Cleaning vent (patented design)
产品优势 7 专利清洗孔设计

It is easy to keep the pouch clean using the cleaning vent (patented design).


专利设计的清洗孔，使用方便，使袋体保持洁净。



Advantage 6 Smooth and washable
产品优势 6 柔滑，可清洗

The special water-repellent nonwoven feels smoothness and can be cleaned up.


特制的拒水无纺布，柔滑，可清洗。

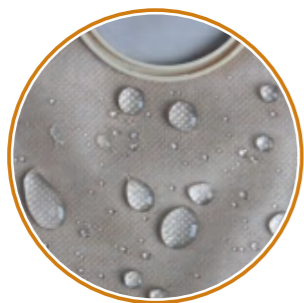

Advantage 8 Velcro design
产品优势 8 粘扣式设计

This velcro design for pouches makes the close tight and firm, but the pouches are still easy to open and be discharged.

粘扣式设计的开口袋，使开口处封闭严密、牢固，又易开启，排放。

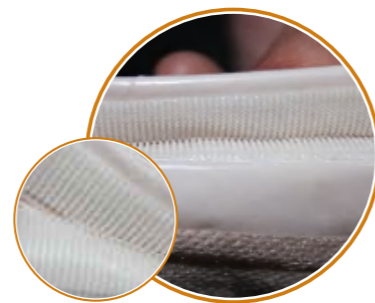


User Guide For One-piece Ostomy Pouch
一件式造口袋的使用方法与技巧

1. Clean the skin around the stoma with tepid water or normal saline from inside to outside, and dry it.
用温水或生理盐水将造口周围皮肤由内向外洗净擦干。



2. Cut the hole on skin barrier according to actual size and shape of the stoma.
底盘已预先开有一个孔，患者可根据造口大小及形状在底盘上剪孔。

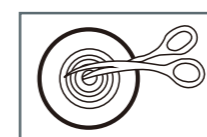


3. To prevent outflow of excreta during sticking, a gauze pad can be temporarily applied on the stoma.
为了防止在粘贴过程中排泄物外流，可在造口处暂时敷上纱布垫。



4. Open the clamp of ostomy pouch.
打开造口袋夹。




5. Close the ostomy pouch with the clamp.
使用前用袋子夹夹住造口袋开口处。

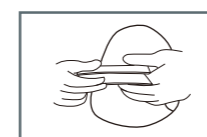

6. Peel off protective release paper from skin barrier.
撕去底盘上的离型保护纸。



7. While sticking, let air into the pouch to prevent pouch wall from sticking, then hold the pouch with both hands to put the skin barrier with the hole cut out on the stoma and press it from bottom up to make it completely conglutinate with the skin. It is suggested to press it evenly for several minutes to strengthen adhesion ability.

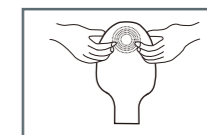

安装袋子时，先向袋内吹小量空气，以防袋壁粘贴，然后双手扶袋将剪好孔的胶盘对准造口部位，从下往上逐步按压，使之与皮肤完全平整粘合，最好均匀按压数分钟以加强粘附能力。

8. Open the pouch end to let the excretion into the toilet. After excretion, clean and dry the pouch as shown for next use.



排便时打开袋尾，袋尾对准便池，排便后如图所示冲洗袋尾，擦干，关闭袋尾重复使用。

9. When peeling off the skin barrier, press the skin with one hand and remove the skin barrier slowly with the other hand. To avoid skin damage, it is forbidden to peel the skin barrier ruggedly. Tepid water can be applied for a few minutes in case of hard peeling.



剥离粘胶时，一手按住皮肤，一手慢慢剥离，切不可强拉硬扯以免造成皮肤损伤。若剥离困难时，可用温水浸润数分钟即可轻松剥离。

Caution 注意事项

- It is used for patient with stoma.
- Skin barrier for this pouch is 5-7 days at normal condition. Prolong use shall affect the adhesive ability, which will probably cause damage to the skin.
- If the pouch is full of air, please exhaust it with pinholes at the top of the pouch. Innomed activated carbon filter is recommended to remove odor.
- Replaced pouches cannot be recycled.
- To protect environment, replaced pouches cannot be directly flushed into toilets and sewers, and cannot be discarded at will.
- Contraindications: patients, allergic to adhesive, should use it carefully.



- 本产品适用于结、回肠造口患者接装排泄物使用。
- 本产品配置的“胶盘”正常情况下使用期限为5-7天，延期使用会影响粘贴效果，对皮肤不利。
- 袋内若充满气体，请在袋体上端用针孔排气，建议使用Innomed活性炭过滤片，可去除异味。
- 换下的造口袋不能再使用。
- 换下的造口袋不能直接冲入厕所及下水道，不能随意丢弃，注意环保。
- 禁忌症，皮肤对粘胶过敏者慎用。

User Guide For Two-piece Ostomy Pouch

二件式造口袋的使用方法技巧


1. Clean the skin around the stoma with tepid water or normal saline from inside to outside, and dry it.

用温水或生理盐水将造口周围皮肤由内向外洗净擦干。




2. Cut the hole on skin barrier according to actual size and shape of the stoma.

底盘已预先开有一个孔，患者可根据造口大小及形状在底盘上剪孔。

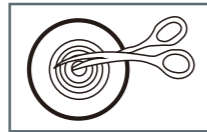


3. To prevent outflow of excreta during sticking, a gauze pad can be temporarily applied on the stoma.

为了防止在粘贴过程中排泄液外流，可在造口处暂时敷上纱布垫。

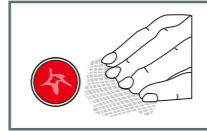

4. Peel off protective release paper from skin barrier, hold the skin barrier with both hands to put it on the hole cut out at the stoma and press it from bottom up to make it completely conglutinate with the skin. It is suggested to press it evenly for several minutes to strengthen adhesion ability.

撕去底盘上的离型保护纸，用双手持盘将剪好孔的胶盘对准造口部位，从下往上逐步按压，使之与皮肤完全平整粘合，最好均匀按压数分钟以加强粘附能力。

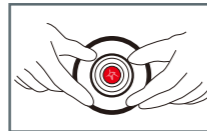

5. After sticking the skin barrier, put the plastic ring on the skin barrier ring, and press the ring with figures from bottom up. Slight sound of "click" shall be heard to confirm that the pouch has been connected with the skin barrier successfully. And pull the pouch to check whether it is a secure connection.

粘好底盘后，将造口袋塑料环对准底盘卡环，手指沿着卡环由下向上压紧，当听见轻轻的“咔嗒”一声，说明袋子已安全地装在底盘上，然后拉拉袋子检查是否牢靠。



6. When removing the pouch, pinch the wing on the connecting ring and pull the pouch away from the skin barrier.

拆卸袋子时，捏住连接环上扶翼将袋子拉离底盘即可。



7. When peeling the skin barrier, press the skin with one hand and remove the skin barrier slowly with the other hand. To avoid skin damage, it is forbidden to peel the skin barrier ruggedly. Tepid water can be applied for a few minutes in case of hard peeling.

剥离粘胶时，一手按住皮肤，一手慢慢剥离，切不可强拉硬扯以免造成皮肤损伤。若剥离困难时，可用温水浸润数分钟即可轻松剥离。



Caution 注意事项


- Before sticking hydrocolloid skin barrier, the skin around the stoma should be clean and dry, and standing or lying position is recommended to keep the skin flat. It is better to press for several minutes evenly after sticking to enhance the adhesive ability.
- Please shave the body hair if there is too much around the stoma, otherwise, the adhesive ability could be affected or folliculitis may occur.
- Please clean the stoma from inside to outside with tepid water instead of disinfectant, soap or oiliness detergent.
- Please be noted, the hole of skin barrier should be 1 - 1.5 mm larger than the stoma, or else, mucosal bleeding or skin maceration irritation could be caused easily.
- If there is swelling for the ostomy pouch frequently, it indicates that there is too much intestinal gas or inefficient gas exhausting. Please make a few small pinholes on the inside of the pouch for adding activated carbon filter.
- The ostomy pouch shall be stored in dry room at the temperature of 15 - 25℃ instead of moist room at high temperature (≥40℃) nor low-temperature facilities such as refrigerator.
- After use, the ostomy pouch shall be wrapped in a clean pouch and put into trash can. The pouch shall not be thrown into toilet to avoid sewer blocking.

- 粘贴造口袋水胶体盘时，首先要保证造口周围皮肤的清洁干爽，最好采用立位或卧位使造口周围皮肤保持平整。平整粘贴后最好均匀按压数分钟以加强粘附力。
- 如造口周围体毛过多、过密者应先将体毛剔除，否则会影响粘贴效果或易导致毛囊炎的发生。
- 清洗造口周围皮肤时不能使用消毒液、肥皂或甚至油性清洁剂，只用温水由造口内圈向外彻底擦洗干净即可。
- 在裁剪粘贴底盘时注意剪空应比造口大1-1.5mm，不能太大或太小，否则易造成黏膜出血或皮肤浸渍刺激。
- 如在造口袋使用中经常充气鼓起，说明场内气体过多或排气不畅，可在造口袋内侧上方用针孔扎几个小孔，补贴活性炭过滤片即可。
- 造口袋应储存于干爽、室温环境（15-25℃），不宜放置在高温（≥40℃）潮湿或冰箱等低温设施内。
- 造口袋使用完毕后应用清洁袋包好扔到垃圾桶内，不能冲入厕所以免造成下水道堵塞。



One-Piece Ostomy Pouch

一件式造口袋

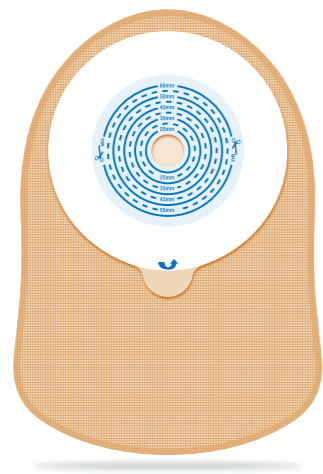

One-Piece Closed Pouch 一件式闭口袋

PE Foam Skin Barrier


- ✔ Multilayer High Barrier Co-extrusion Membrane.
- ✔ The foam skin barrier has good adhesive ability with low sensitization, and is not easy to roll up.
- ✔ The pouch is small, soft, comfortable and concealed.
- ✔ It is easy to use at low cost.
- ✔ It is suitable for patients having shaped feces or regular excretion.

PE泡棉底盘

- ✔ 高阻隔性多层共挤薄膜
- ✔ 泡棉胶盘，粘帖性好，不易翘边，低致敏性
- ✔ 造型小巧、柔软，舒适隐蔽
- ✔ 使用方便，价格低廉
- ✔ 适用于粪便成型或排泄物规律的造口人士

Product Selection

Reorder No.	Description	Quantity
4002	1-3/4" (44mm)	40/10/Cs
4001	2-1/4" (57mm)	40/10/Cs

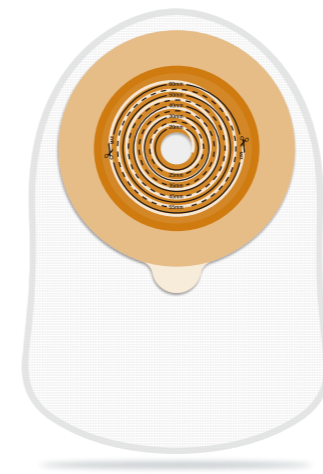

One-Piece Open Pouch (Threaded) 一件式闭口袋 (螺纹型)

Hydrocolloid Skin Barrier, Threaded


- ✔ which is a special structure designed for high standard safety and protection.
- ✔ The skin barrier has good initial and lasting adhesive ability.
- ✔ Comfortable, noiselessly and moisture wicking special nonwoven is applied on the pouch for skin side.
- ✔ Transparent membrane is used at outer side, which is convenient to observe and clean.

水胶体底盘，螺纹型

- ✔ 螺纹型水胶体底盘，更高标准地体现安全性和保护性所设计的特殊结构
- ✔ 初粘性、持粘性好
- ✔ 贴合皮肤面为特制无纺布，舒适、吸汗且无声
- ✔ 外侧面为透明膜材质，利于观察及清洗

Product Selection

Reorder No.	Description	Quantity
6001	1-3/4" (44mm)	40/10/Cs
6002	2-1/4" (57mm)	40/10/Cs

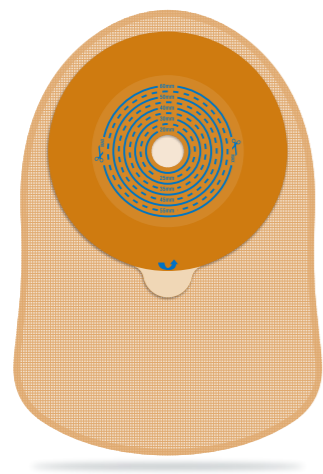

One-Piece Closed Pouch 一件式闭口袋

Hydrocolloid Skin Barrier


- ✔ Multilayer high barrier co-extrusion membrane.
- ✔ Which has good adhesive ability for long period and no sensitization, and is comfortable and skin-friendly.
- ✔ The pouch is small, soft, comfortable and concealed.
- ✔ It is easy to use at low cost.
- ✔ It is suitable for patients having shaped feces or regular excretion.

水胶体底盘

- ✔ 高阻隔性多层共挤薄膜
- ✔ 水胶体胶盘，粘帖性好，粘帖时间长，舒适、对皮肤友好、无致敏
- ✔ 造型小巧、柔软，舒适隐蔽
- ✔ 使用方便，价格低廉
- ✔ 适用于粪便成型或排泄物规律的造口人士

Product Selection

Reorder No.	Description	Quantity
4003	1-3/4" (44mm)	40/10/Cs
4004	2-1/4" (57mm)	40/10/Cs

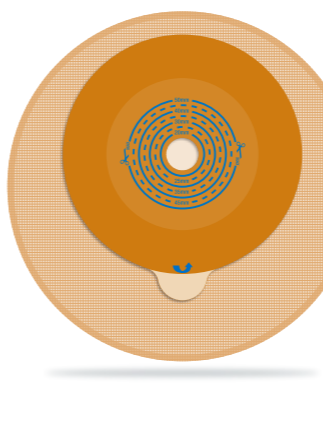

Mini Pouch 迷你袋

Hydrocolloid Skin Barrier

- ✔ The hydrocolloid skin barrier has good adhesive ability with no sensitization, and is comfortable and skin-friendly.
- ✔ It is equipped with a built-in carbon filter.
- ✔ Soft and opaque double-side non-woven fabric lining is used.
- ✔ The pouch is small, concealed and easy to use.


水胶体底盘

- ✔ 水胶体胶盘，粘帖性好，舒适，对皮肤友好、无致敏
- ✔ 内置式过滤碳片
- ✔ 双面无纺布被衬柔软，不透明
- ✔ 小巧、隐蔽，使用方便

Product Selection

Reorder No.	Description	Quantity
4013	1-3/4" (44mm)	40/10/Cs





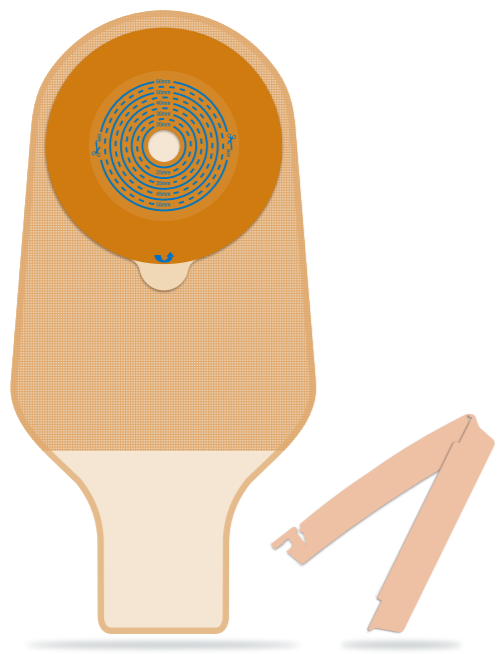

One-Piece Open Pouch (Clip) 一件式开口袋 (夹扣式)

- | | |
|---|--|
| <p>Hydrocolloid Skin Barrier</p> <ul style="list-style-type: none"> ✔ Multilayer high barrier co-extrusion membrane. ✔ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5 - 7 days) It is comfortable and skin-friendly with no sensitization. ✔ The built-in carbon filter can avoid swelling and odor. ✔ Open bag, easy to discharge and clean ✔ Easy operation and use | <p>水胶体底盘</p> <ul style="list-style-type: none"> ✔ 高阻隔性多层共挤薄膜 ✔ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏 ✔ 内置式过滤碳片，避免胀袋，防止异味 ✔ 开口袋，方便排放、清洗 ✔ 操作使用方便 |
|---|--|



Reorder No.	Description	Quantity
4005	1-3/4" (44mm)	40/10/Cs
4006	2-1/4" (57mm)	40/10/Cs
4007	2-3/8" (60mm)	40/10/Cs
4012	2-3/4" (70mm)	40/10/Cs

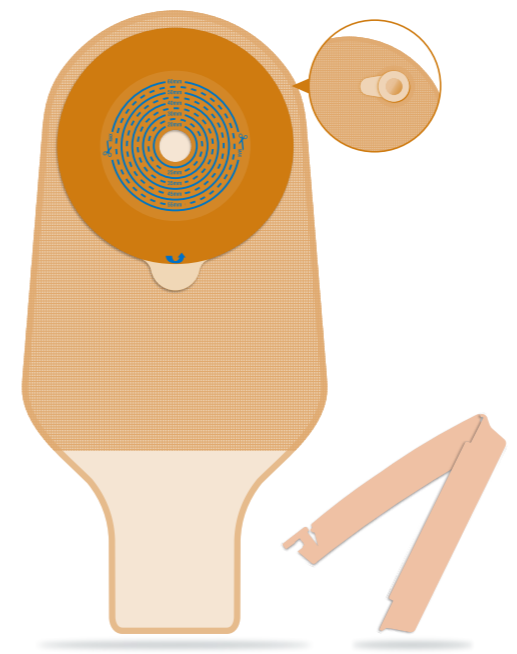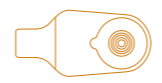

One-Piece Open Pouch (Vent Hole) 一件式开口袋 (冲洗孔式)

- | | |
|---|--|
| <p>Hydrocolloid Skin Barrier</p> <ul style="list-style-type: none"> ✔ Multilayer high barrier co-extrusion membrane. ✔ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5 - 7 days) It is comfortable and skin-friendly with no sensitization. ✔ The built-in carbon filter can avoid swelling and odor. ✔ Open bag, easy to discharge and clean ✔ Easy operation and use | <p>水胶体底盘</p> <ul style="list-style-type: none"> ✔ 高阻隔性多层共挤薄膜 ✔ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏 ✔ 内置式过滤碳片，避免胀袋，防止异味 ✔ 开口袋，方便排放、清洗 ✔ 操作使用方便 |
|---|--|



Reorder No.	Description	Quantity
4005I	1-3/4" (44mm)	40/10/Cs
4006I	2-1/4" (57mm)	40/10/Cs
4007I	2-3/8" (60mm)	40/10/Cs
4012I	2-3/4" (70mm)	40/10/Cs

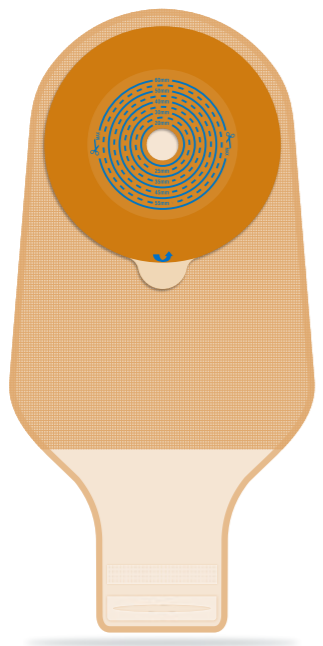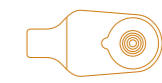

One-Piece Open Pouch (Velcro) 一件式开口袋 (魔术贴式)

- | | |
|---|--|
| <p>Hydrocolloid Skin Barrier</p> <ul style="list-style-type: none"> ✔ Multilayer high barrier co-extrusion membrane. ✔ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5 - 7 days) It is comfortable and skin-friendly with no sensitization. ✔ The built-in carbon filter can avoid swelling and odor. ✔ Open bag, easy to discharge and clean ✔ Easy operation and use | <p>水胶体底盘</p> <ul style="list-style-type: none"> ✔ 高阻隔性多层共挤薄膜 ✔ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏 ✔ 内置式过滤碳片，避免胀袋，防止异味 ✔ 开口袋，方便排放、清洗 ✔ 操作使用方便 |
|---|--|


Reorder No.	Description	Quantity
4005V	1-3/4" (44mm)	40/10/Cs
4006V	2-1/4" (57mm)	40/10/Cs
4007V	2-3/8" (60mm)	40/10/Cs
4012V	2-3/4" (70mm)	40/10/Cs

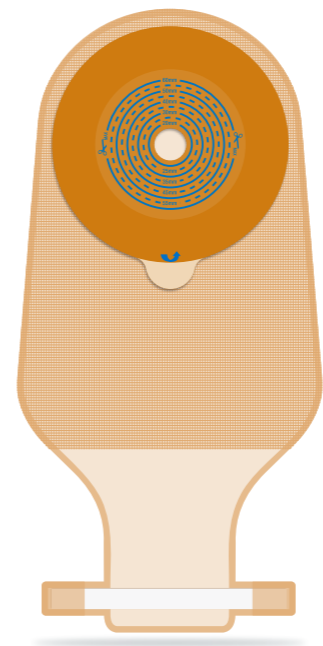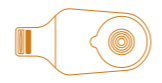


One-Piece Open Pouch (Sticky Aluminum Strip) 一件式开口袋 (内置铝条式)

- | | |
|---|--|
| <p>Hydrocolloid Skin Barrier</p> <ul style="list-style-type: none"> ✔ Multilayer high barrier co-extrusion membrane. ✔ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5 - 7 days) It is comfortable and skin-friendly with no sensitization. ✔ The built-in carbon filter can avoid swelling and odor. ✔ Open bag, easy to discharge and clean ✔ Easy operation and use | <p>水胶体底盘</p> <ul style="list-style-type: none"> ✔ 高阻隔性多层共挤薄膜 ✔ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏 ✔ 内置式过滤碳片，避免胀袋，防止异味 ✔ 开口袋，方便排放、清洗 ✔ 操作使用方便 |
|---|--|

Reorder No.	Description	Quantity
4005T	1-3/4" (44mm)	40/10/Cs
4006T	2-1/4" (57mm)	40/10/Cs
4007T	2-3/8" (60mm)	40/10/Cs
4012T	2-3/4" (70mm)	40/10/Cs

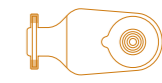

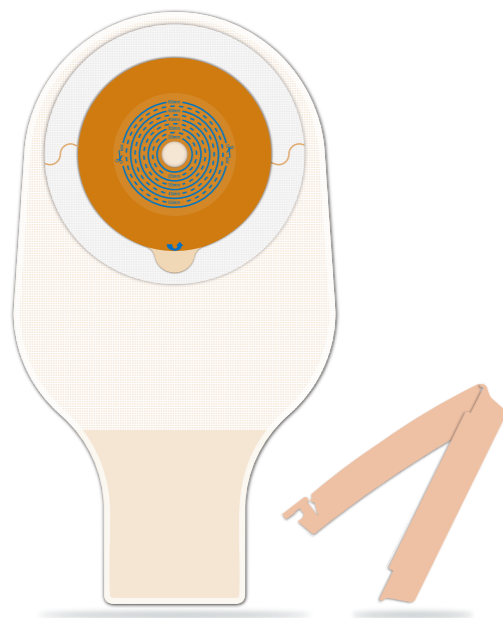

One-Piece Open Pouch (Clip)

一件式开口袋 (夹扣式)

It is equipped with hydrocolloid skin barrier for inner ring and nonwoven adhesive tape for outer ring


- ✓ Multilayer high barrier co-extrusion membrane.
- ✓ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days) It is comfortable and skin-friendly with no sensitization.
- ✓ As nonwoven is used for outer ring, the pouch can be fixed more firmly and anti-leakage.
- ✓ The built-in carbon filter can avoid swelling and odor.
- ✓ Open bag, easy to discharge and clean.
- ✓ Easy operation and use.

内圈水胶体胶盘，外圈无纺布胶布

- ✓ 高阻隔性多层共挤薄膜
- ✓ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏
- ✓ 外圈为无纺布，固定更牢固，同时防渗漏
- ✓ 内置式过滤碳片，避免胀袋，防止异味
- ✓ 开口袋，方便排放，清洗
- ✓ 操作使用方便

Product Selection

Reorder No.	Description	Quantity
4008	1-3/4" (44mm)	40/10/Cs
4009	2-1/4" (57mm)	40/10/Cs

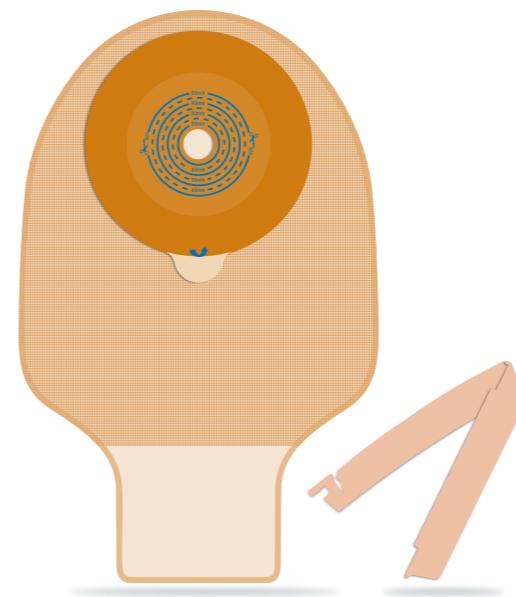

One-Piece Open Pouch For Children

一件式开口儿童袋

Hydrocolloid Skin Barrier


- ✓ The hydrocolloid skin barrier has good adhesive ability with no sensitization, and is comfortable and skin-friendly.
- ✓ Non-woven or PE membrane lining is used at inner side.
- ✓ And the pouch is suitable for pediatric stoma.

水胶体底盘

- ✓ 水胶体胶盘，粘贴性好，舒适，对皮肤友好、无致敏
- ✓ 内侧被衬无纺布或PE网膜
- ✓ 适用于小儿造口

Product Selection

Reorder No.	Description	Quantity
4014	1-1/4" (32mm)	40/10/Cs
4015	1-3/4" (44mm)	40/10/Cs

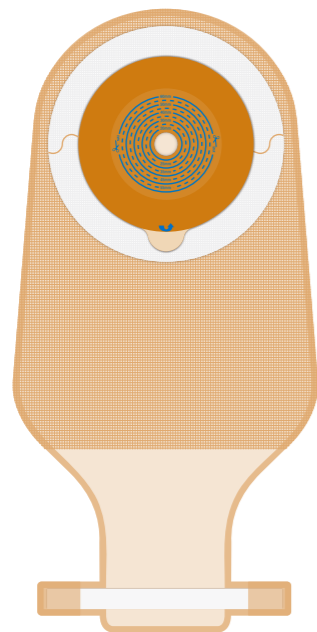

One-Piece Open Pouch (Sticky Aluminum Strip)

一件式开口袋 (内置铝条式)

It is equipped with hydrocolloid skin barrier for inner ring and nonwoven adhesive tape for outer ring


- ✓ Multilayer high barrier co-extrusion membrane.
- ✓ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days) It is comfortable and skin-friendly with no sensitization.
- ✓ As nonwoven is used for outer ring, the pouch can be fixed more firmly and anti-leakage.
- ✓ The built-in carbon filter can avoid swelling and odor.
- ✓ Open bag, easy to discharge and clean.
- ✓ Easy operation and use.

内圈水胶体胶盘，外圈无纺布胶布

- ✓ 高阻隔性多层共挤薄膜
- ✓ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏
- ✓ 外圈为无纺布，固定更牢固，同时防渗漏
- ✓ 内置式过滤碳片，避免胀袋，防止异味
- ✓ 开口袋，方便排放，清洗
- ✓ 操作使用方便

Product Selection

Reorder No.	Description	Quantity
4008T	1-3/4" (44mm)	40/10/Cs
4009T	2-1/4" (57mm)	40/10/Cs

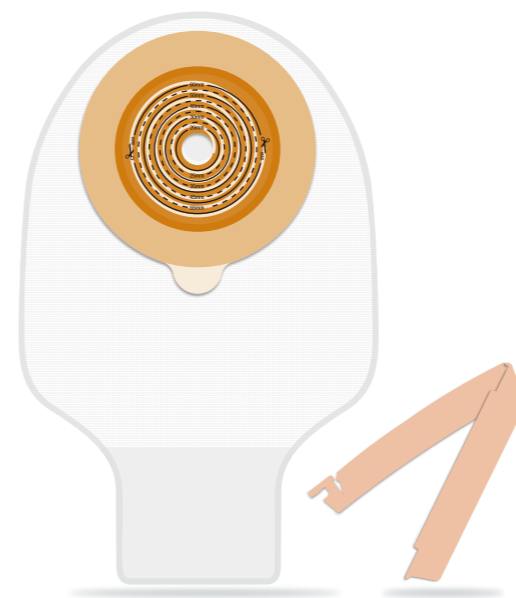

One-Piece Open Pouch For Children (Threaded)

一件式开口儿童袋 (螺纹型)

Hydrocolloid Skin Barrier, Threaded

- ✓ which is a special structure designed for high standard safety and protection.
- ✓ Comfortable, noiselessly and moisture wicking special nonwoven is applied on the pouch for skin side.
- ✓ Transparent membrane is used at outer side, which is convenient to observe and clean.
- ✓ And the pouch is suitable for pediatric stoma.


水胶体底盘，螺纹型

- ✓ 螺纹型水胶体底盘，更高标准地体现安全性和保护性所设计的特殊结构
- ✓ 贴合皮肤面为特制无纺布，舒适、吸汗且无声
- ✓ 外侧面为透明膜材质，利于观察及清洗
- ✓ 适用于小儿造口

Product Selection

Reorder No.	Description	Quantity
6014	1-1/4" (32mm)	40/10/Cs
6015	1-3/4" (44mm)	40/10/Cs



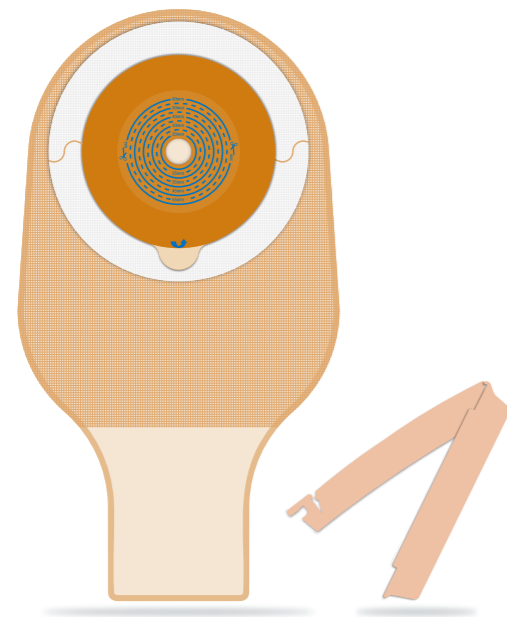

One-Piece Open Pouch (Clip)

一件式开口袋 (夹扣式)


It is equipped with hydrocolloid skin barrier for inner ring and nonwoven adhesive tape for outer ring

内圈水胶体胶盘，外圈无纺布胶布

- ✔ Multilayer high barrier co-extrusion membrane.
 - ✔ The hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days) It is comfortable and skin-friendly with no sensitization.
 - ✔ As nonwoven is used for outer ring, the pouch can be fixed more firmly and anti-leakage.
 - ✔ The built-in carbon filter can avoid swelling and odor.
 - ✔ Open bag, easy to discharge and clean.
 - ✔ Easy operation and use.
- ✔ 高阻隔性多层共挤薄膜
 - ✔ 水胶体胶盘，粘贴性好，粘贴时间长，可达5-7天。舒适、对皮肤友好、无致敏
 - ✔ 外圈为无纺布，固定更牢固，同时防渗漏
 - ✔ 内置式过滤碳片，避免胀袋，防止异味
 - ✔ 开口袋，方便排放，清洗
 - ✔ 操作使用方便

Product Selection

Reorder No.	Description	Quantity
5009	2-1/4" (57mm)	40/10/Cs
5009C	2-1/4" (57mm)	40/10/Cs

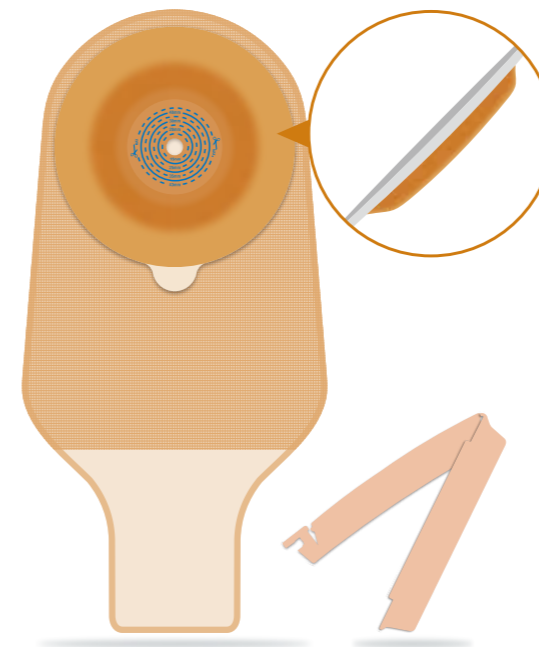

One-Piece Open Pouch (Convex Skin Barrier)

一件式开口袋 (凸面底盘)


Hydrocolloid Skin Barrier

水胶体底盘

- ✔ The convex hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days) to prevent leakage effectively. It is soft, comfortable and skin-friendly with no sensitization, and has good absorption for exudate.
 - ✔ Open bag, easy to discharge and clean.
 - ✔ Easy operation and use.
- ✔ 中央凸出设计的水胶体底盘，有效防止渗漏，粘贴性好，粘贴时间长，一般可达5-7天。吸收渗液好，柔软、舒适，对皮肤友好，无致敏。
 - ✔ 开口袋，方便排放，清洗
 - ✔ 操作使用方便

Product Selection

Reorder No.	Description	Quantity
4063	1-1/4" (33mm)	40/10/Cs
4064	1-3/4" (43mm)	40/10/Cs

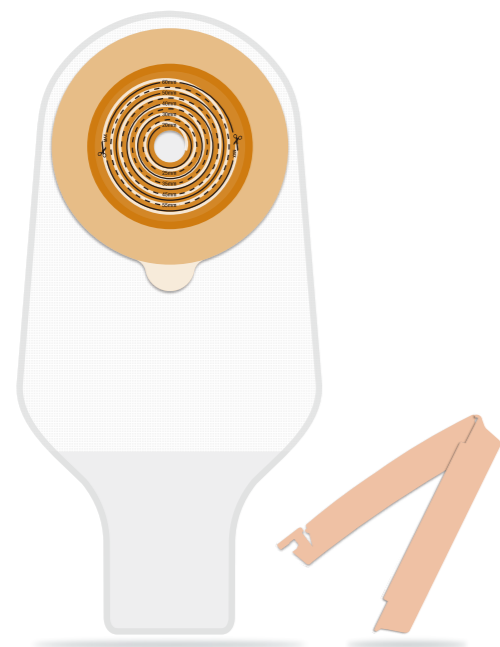

One-Piece Open Pouch (Threaded)

一件式开口袋 (螺纹型)

Hydrocolloid Skin Barrier, Threaded

水胶体底盘，螺纹型

- ✔ Which is a special structure designed for high standard safety and protection.
 - ✔ The skin barrier has good initial and lasting adhesive ability.
 - ✔ Comfortable, noiselessly and moisture wicking special nonwoven is applied on the pouch for skin side.
 - ✔ Transparent membrane is used at outer side, which is convenient to observe and clean.
- ✔ 螺纹型水胶体底盘，更高标准地体现安全性和保护性所设计的特殊结构
 - ✔ 初粘性、持粘性好
 - ✔ 贴合皮肤面为特制无纺布，舒适、吸汗且无声
 - ✔ 外侧面为透明膜材质，利于观察及清洗



Product Selection

Reorder No.	Description	Quantity
6005	1-3/4" (44mm)	40/10/Cs
6006	2-1/4" (57mm)	40/10/Cs
6007	2-3/8" (60mm)	40/10/Cs
6012	2-3/4" (70mm)	40/10/Cs





Two-Piece Ostomy Pouch
二件式造口袋

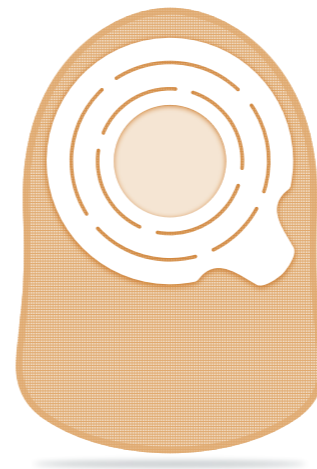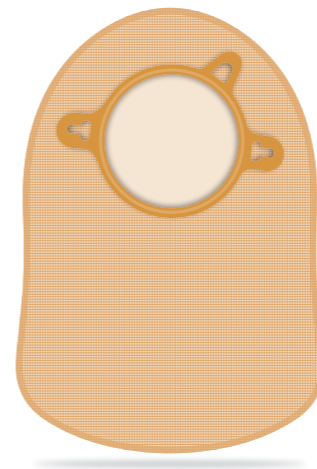

Two-Piece Closed Pouch 二件式闭口袋

Embedded

- ✔ Soft and comfortable.
- ✔ Embedded ring.
- ✔ It is small and easy to use.

嵌入式

- ✔ 柔软、舒适
- ✔ 嵌入式卡环
- ✔ 小巧，使用方便

Product Selection

Reorder No.	Description	Quantity
4016	1-3/4" (44mm)	40/10/Cs
4017	2-1/4" (57mm)	40/10/Cs



Two-Piece Closed Pouch 二件式闭口袋

Adhesive

- ✔ The hydrocolloid skin barrier has good adhesive ability with no sensitization, and is comfortable and skin-friendly.
- ✔ The adhesive design without ring, and is easy to use with no influence on movement.


粘贴式

- ✔ 水胶体胶盘，粘贴性好，舒适，对皮肤友好，无致敏
- ✔ 粘贴式设计，无卡环，使用更方便，活动不受影响

Product Selection

Reorder No.	Description	Quantity
4018	1-3/4" (44mm)	40/10/Cs
4019	2-1/4" (57mm)	40/10/Cs



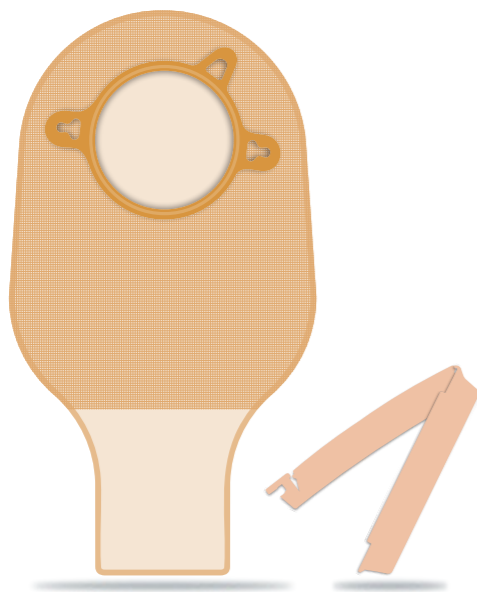


Two-Piece Open Pouch (Clip) 二件式开口袋 (夹扣式)

- | | |
|---|--------------|
| Embedded | 嵌入式 |
| ✔ Soft and comfortable. | ✔ 柔软、舒适 |
| ✔ It is equipped with nonwoven or PE membrane lining. | ✔ 内衬无纺布或PE网膜 |
| ✔ Flexibloring. | ✔ 卡环易扣合 |
| ✔ Built-in carbon filter. | ✔ 内置过滤碳片 |

Product Selection

Reorder No.	Description	Quantity
4020	1-3/4" (44mm)	40/10/Cs
4021	2-1/4" (57mm)	40/10/Cs
4022	2-3/8" (60mm)	40/10/Cs

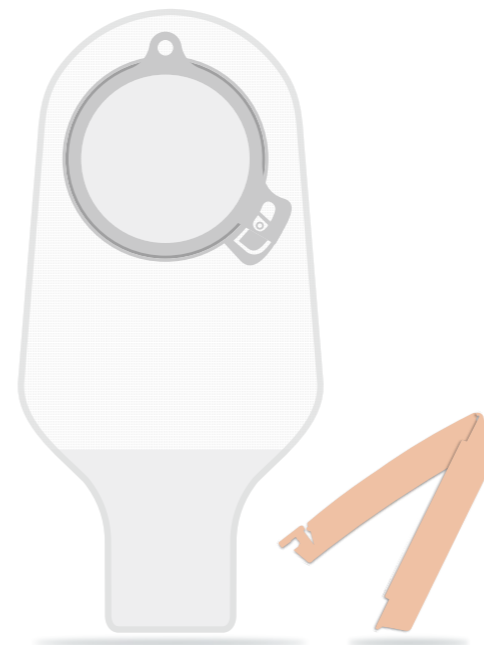


Two-Piece Open Pouch (Mechanical Fit) 二件式开口袋 (机械扣合式)

- | | |
|--|---------------------------------------|
| Mechanical Fit | 机械扣合式 |
| ✔ The pouch uses mechanical fit, obtaining double upgrading in safety and comfort. | ✔ 安全性、舒适度双重升级 |
| ✔ Through the mechanical fit system, the pouch can be secured on the skin barrier. | ✔ 通过机械扣合系统，造口袋可以安全地固定在底盘上 |
| ✔ Can be turned to right position without removing from the skin barrier before being locked. | ✔ 在造口袋锁定之前，不用从底盘上取下造口袋就可以将造口袋转动到合适的位置 |
| ✔ It is easy to remove and replace frequently comparing with the ordinary adhesive skin barrier. | ✔ 拆卸方便，相较于普通黏胶底盘，造口袋可以更频繁地进行更换 |

Product Selection

Reorder No.	Description	Quantity
6020	1-3/4" (44mm)	40/10/Cs
6021	2-1/4" (57mm)	40/10/Cs
6022	2-3/8" (60mm)	40/10/Cs

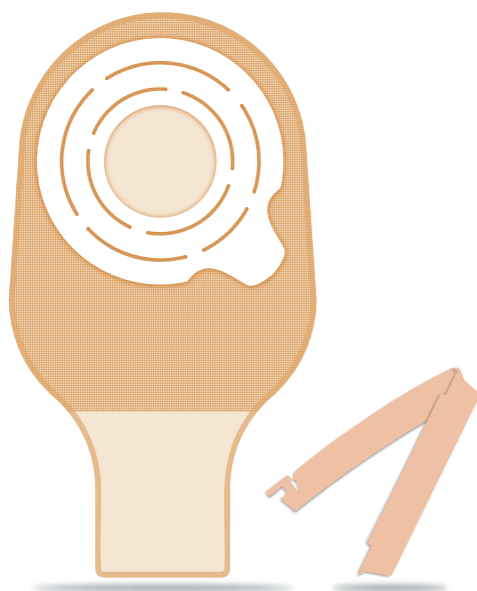


Two-Piece Open Pouch (Clip) 二件式开口袋 (夹扣式)

- | | |
|---|--------------------------|
| Adhesive | 粘贴式 |
| ✔ Soft and comfortable. | ✔ 柔软、舒适 |
| ✔ It is equipped with nonwoven or PE membrane lining. | ✔ 内衬无纺布或PE网膜 |
| ✔ The adhesive design includes no ring, and is easy to use with no influence on movement. | ✔ 粘贴式设计，无卡环，使用更方便，活动不受影响 |
| ✔ Built-in carbon filter. | ✔ 内置过滤碳片 |

Product Selection

Reorder No.	Description	Quantity
4023	1-3/4" (44mm)	40/10/Cs
4024	2-1/4" (57mm)	40/10/Cs
4025	2-3/8" (60mm)	40/10/Cs

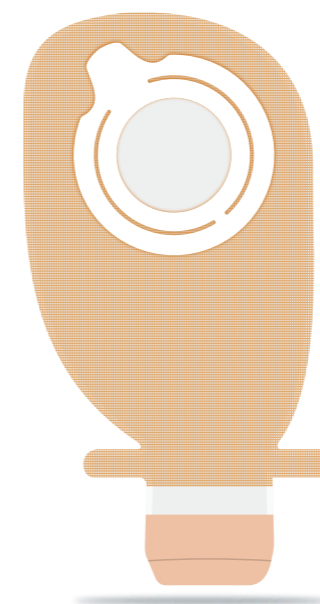


Two-Piece Drainable Pouch 二件式开口袋

- | | |
|--|-----------------------|
| Adhesive | 粘贴式 |
| ✔ Soft and comfortable. | ✔ 柔软、舒适 |
| ✔ It is equipped with non-woven or PE membrane lining. | ✔ 内衬无纺布或PE网膜 |
| ✔ The pouch is simple with no ring and more easy to use with no influence on movement. | ✔ 简约无卡环，使用更方便，活动不受影响 |
| ✔ The new design for opening makes the pouch easy to discharge, clean and use. | ✔ 新型开口设计，方便排放、清洗，操作简单 |

Product Selection

Reorder No.	Description	Quantity
6023	1-3/4" (44mm)	40/10/Cs
6024	2-1/4" (57mm)	40/10/Cs
6025	2-3/8" (60mm)	40/10/Cs

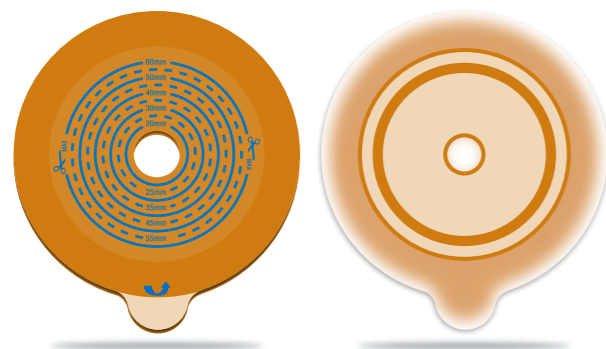

Two-Piece Skin Barrier 二件式底盘

Hydrocolloid Skin Barrier (Embedded)


- ✔ Specially designed hydrocolloid skin barrier has good adhesive ability with no sensitization for 5-7 days, and is comfortable and skin-friendly, but not easy to roll up.

水胶体底盘 (嵌入式)

- ✔ 特殊设计的水胶体底盘，粘贴性好，不易翘边，舒适，对皮肤友好，无致敏，使用时间一般可达5-7天

Product Selection

Reorder No.	Description	Quantity
4026	1-3/4" (44mm)	40/5/Cs
4027	2-1/4" (57mm)	40/5/Cs
4028	2-3/8" (60mm)	40/5/Cs

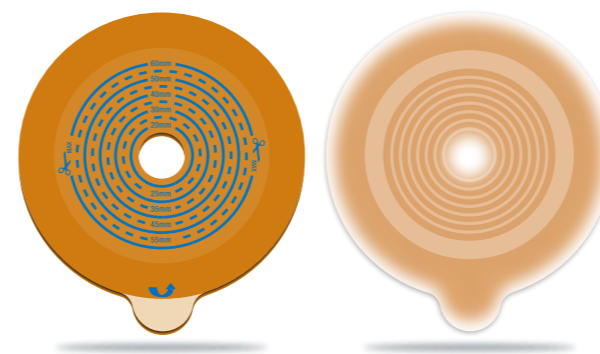

Two-Piece Skin Barrier 二件式底盘

Hydrocolloid Skin Barrier (Adhesive)


- ✔ Special designed hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days), and is comfortable and skin-friendly with no sensitization. The adhesive ring in no-ring design is treated by special process for firmly fix, and is easy to use.

水胶体底盘 (粘贴式)

- ✔ 特殊设计的水胶体底盘，粘贴性好，粘贴时间长，一般可达5-7天。舒适，对皮肤友好，无致敏。无卡环设计，粘贴圈经特殊工艺处理，粘贴牢固，使用方便

Product Selection

Reorder No.	Description	Quantity
4035	1-3/4" (44mm)	40/5/Cs
4036	2-1/4" (57mm)	40/5/Cs
4037	2-3/8" (60mm)	40/5/Cs

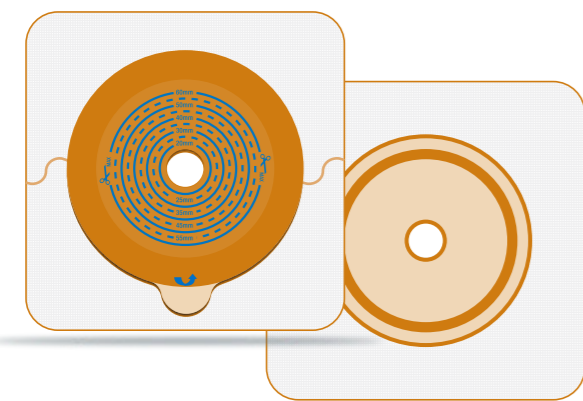

Two-Piece Skin Barrier 二件式底盘

It is equipped with hydrocolloid skin barrier for inner ring and nonwoven adhesive tape for outer ring (Embedded)


- ✔ The hydrocolloid skin barrier has good adhesive ability with no sensitization for long period (up to 5-7 days), and is comfortable and skin-friendly. The pouch can be anti-leakage and fixed more firmly due to nonwoven used for outer ring.

内圈水胶体胶盘，外圈无纺布胶布 (嵌入式)

- ✔ 水胶体底盘，粘贴性好，舒适、对皮肤友好，无致敏，粘贴时间长，一般可达5-7天，外圈为无纺布胶布，固定更牢固，同时防渗漏

Product Selection

Reorder No.	Description	Quantity
4029	1-3/4" (44mm)	40/5/Cs
4030	2-1/4" (57mm)	40/5/Cs
4031	2-3/8" (60mm)	40/5/Cs

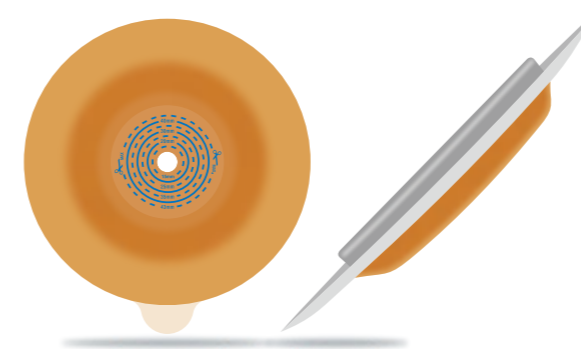

Two-Piece Convex Skin Barrier 二件式凸面底盘


- ✔ The convex hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days) to prevent leakage effectively. It is soft, comfortable and skin-friendly with no sensitization, and has good absorption for exudate. The ring is easy to fit and use.

- ✔ 中央凸出设计的水胶体底盘，有效防止渗漏，粘贴性好，粘贴时间长，一般可达5-7天。吸收渗液好，柔软、舒适，对皮肤友好，无致敏。卡环易扣合，使用方便

Product Selection

Reorder No.	Description	Quantity
4060	1-1/4" (33mm)	40/5/Cs
4061	1-3/4" (43mm)	40/5/Cs



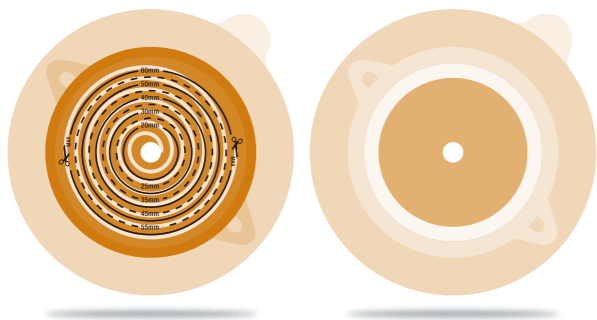

Two-Piece Skin Barrier 二件式底盘

Hydrocolloid Skin Barrier (Threaded)

✔ Threaded hydrocolloid adhesive can provide additional skin protection for high excretion. The pouch is suitable for patients having ileostomy or urostomy, and is safer and more protectable with threaded design comparing with ordinary hydrocolloid skin barrier.

✔ The hydrocolloid skin barrier treated by special process contains good initial and lasting adhesive ability, and is soft, comfortable and skin-friendly with no sensitization.

水胶体底盘 (螺旋型)

✔ 螺旋形黏胶可以在排泄物较多时为皮肤提供额外的保护。适用于排泄物较多的回肠造口或尿路造口人士，螺旋纹设计的安全性和保护性高于普通黏胶底盘。

✔ 特殊工艺水胶体底盘，初粘、持粘佳；舒适，对皮肤友好且无致敏。

Product Selection

Reorder No.	Description	Quantity
6026	1-3/4" (44mm)	40/5/Cs
6027	2-1/4" (57mm)	40/5/Cs
6028	2-3/8" (60mm)	40/5/Cs



Two-Piece Convex Skin Barrier 二件式凸面底盘

Hydrocolloid Skin Barrier (Threaded)

✔ It is suitable for stoma at the same level of the skin or retracted stoma. Additional safety can be provided by applying gentle pressure to the abdomen.

✔ The hydrocolloid skin barrier treated by special process contains good initial and lasting adhesive ability, and is soft, comfortable and skin-friendly with no sensitization.


水胶体底盘 (螺旋型)

✔ 适用于与皮肤平齐或回缩的造口，其通过对腹部施加轻柔的压力，可以提供额外的安全性。


✔ 特殊工艺水胶体底盘，初粘、持粘佳；舒适，对皮肤友好且无致敏。

Product Selection

Reorder No.	Description	Quantity
6060	1-1/4" (33mm)	40/5/Cs
6061	1-3/4" (43mm)	40/5/Cs

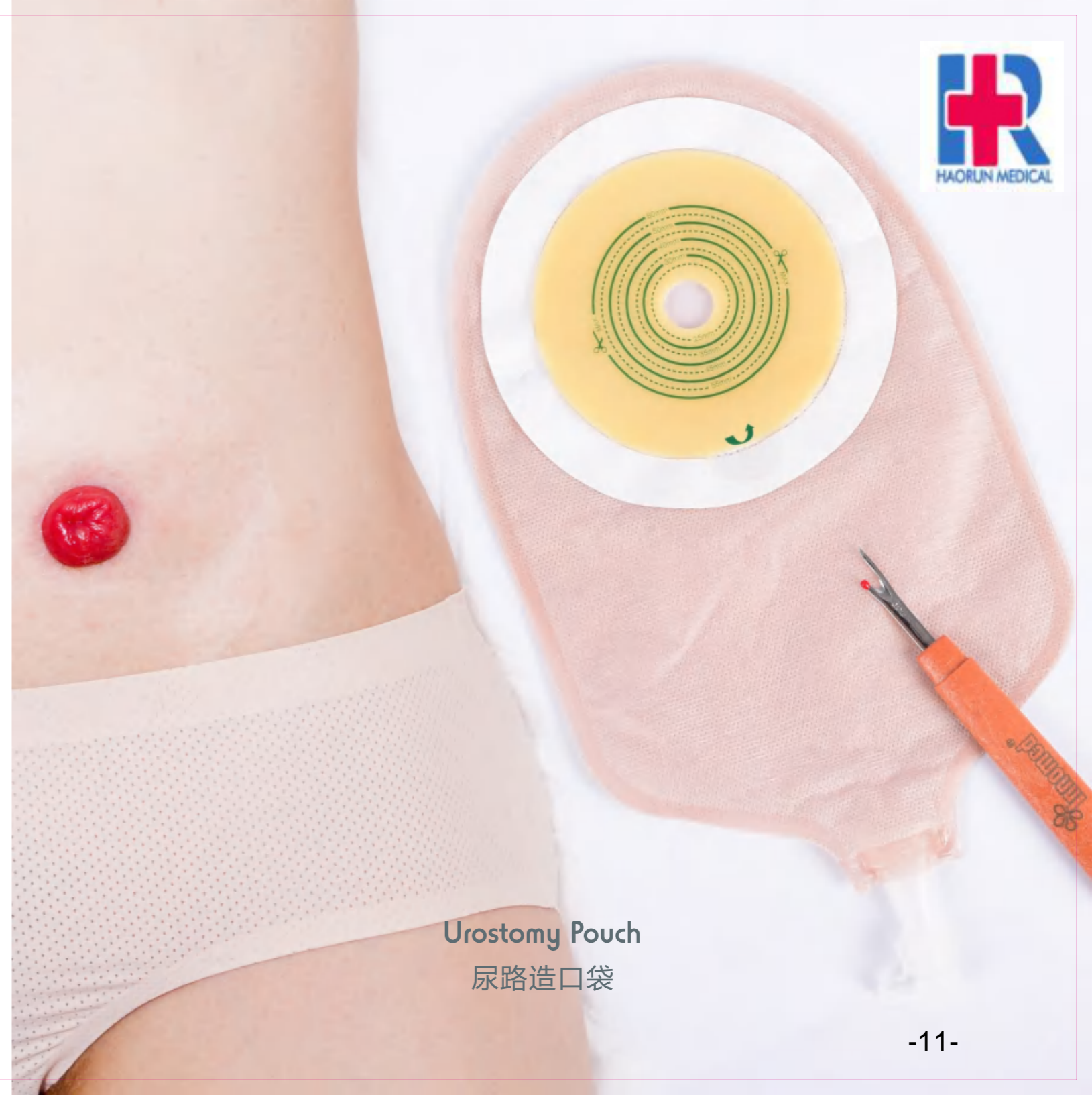

Urostomy Pouch
尿路造口袋

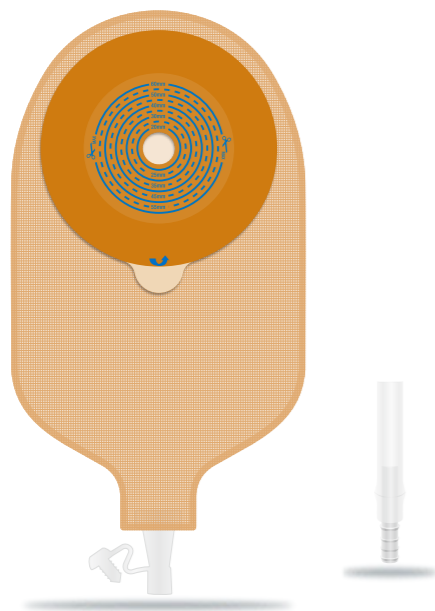

One-Piece Urostomy Pouch 一件式尿路造口袋

Hydrocolloid Skin Barrier

- ✔ Special designed hydrocolloid skin barrier has good adhesive ability for long period (up to 5-7 days), and is comfortable and skin-friendly with no sensitization.
- ✔ Rotary valve design is adopted at the urination orifice. It is quick and convenient to drainage, and it is hygienic and clean.

水胶体底盘

- ✔ 特殊设计的水胶体底盘，粘贴性好，粘贴时间长，一般可达5-7天，舒适，对皮肤友好，无致敏
- ✔ 排尿口处采用旋转式阀门设计，排尿快而方便，且卫生、干净

Product Selection

Reorder No.	Description	Quantity
4038	1-3/4" (44mm)	40/10/Cs
4039	2-1/4" (57mm)	40/10/Cs



One-Piece Urostomy Pouch 一件式尿路造口袋

Hydrocolloid Skin Barrier, Threaded


- ✔ It is equipped with threaded hydrocolloid skin barrier, which is a special structure designed for high standard safety and protection.
- ✔ Transparent membrane is used at outer side, which is convenient to observe and clean.
- ✔ Rotary valve design is adopted at the urination orifice. It is quick and convenient to drainage, and it is hygienic and clean.

水胶体底盘，螺纹型

- ✔ 螺纹型水胶体底盘，更高标准地体现安全性和保护性所设计的特殊结构
- ✔ 外侧面为透明膜材质，利于观察及清洗
- ✔ 排尿口处采用旋转式阀门设计，排尿快而方便，且卫生、干净

Product Selection

Reorder No.	Description	Quantity
6038	1-3/4" (44mm)	40/10/Cs
6039	2-1/4" (57mm)	40/10/Cs

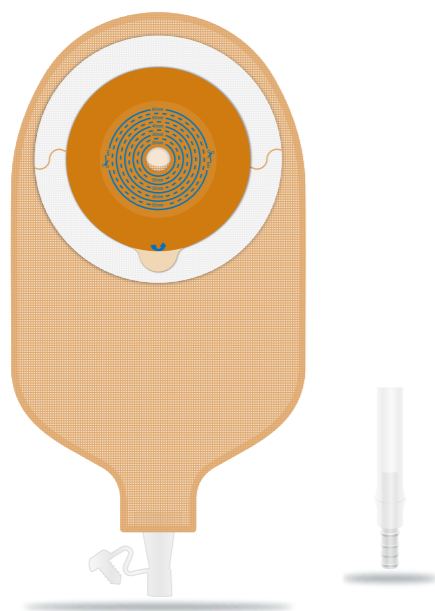

One-Piece Urostomy Pouch 一件式尿路造口袋

It is equipped with hydrocolloid skin barrier for inner ring and nonwoven adhesive tape for outer ring.


- ✔ The hydrocolloid skin barrier has good adhesive ability with no sensitization for long period (up to 5-7 days), and is comfortable and skin-friendly.
- ✔ The pouch can be anti-leakage and fixed more firmly due to nonwoven adhesive tape used for outer ring.
- ✔ Rotary valve design is adopted at the urination orifice. It is quick and convenient to drainage, and it is hygienic and clean.

内圈水胶体胶盘，外圈无纺布胶布

- ✔ 特殊设计的水胶体底盘，粘贴性好，粘贴时间长，一般可达5-7天，舒适，对皮肤友好，无致敏
- ✔ 外圈为无纺布胶布，固定更牢固，同时防渗漏
- ✔ 排尿口处采用旋转式阀门设计，排尿快而方便，且卫生、干净

Product Selection

Reorder No.	Description	Quantity
4040	1-3/4" (44mm)	40/10/Cs
4041	2-1/4" (57mm)	40/10/Cs

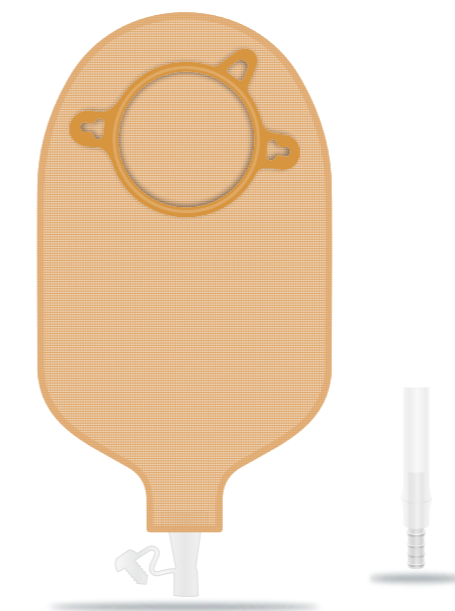

Two-Piece Urostomy Pouch 二件式尿路造口袋

Embedded

- ✔ Soft and comfortable.
- ✔ It is equipped with nonwoven or PE membrane lining.
- ✔ A ring that is easy to fit.
- ✔ Rotary valve design is adopted at the urination orifice. It is quick and convenient to drainage, and it is hygienic and clean.

嵌入式

- ✔ 柔软、舒适
- ✔ 内衬无纺布或PE网膜
- ✔ 卡环易扣合
- ✔ 排尿口处采用旋转式阀门设计，排尿快而方便，且卫生、干净

Product Selection

Reorder No.	Description	Quantity
4044	1-3/4" (44mm)	40/10/Cs
4045	2-1/4" (57mm)	40/10/Cs





Two-Piece Urostomy Pouch 二件式尿路造口袋

- | | |
|--|---|
| <p>Mechanical Fit</p> <ul style="list-style-type: none"> ✔ Through the mechanical fit system, the pouch can be secured on the skin barrier. ✔ Can be turned to right position without removing from the skin barrier before being locked. ✔ Rotary valve design is adopted at the urination orifice. It is quick and convenient to drainage, and it is hygienic and clean. | <p>机械扣合式</p> <ul style="list-style-type: none"> ✔ 通过机械扣合系统，造口袋可以安全地固定在底盘上 ✔ 在造口袋锁定之前，不用从底盘上取下造口袋就可以将造口袋转动到合适的位置 ✔ 排尿口处采用旋转式阀门设计，排尿快而方便，且卫生、干净 |
|--|---|

Product Selection		
Reorder No.	Description	Quantity
6044	1-3/4" (44mm)	40/10/Cs
6045	2-1/4" (57mm)	40/10/Cs



Ostomy Attachments
造口附件




Medical Silicon Ostomy Loop

医用硅胶造口环

- ✓ Patented design
- ✓ Economical, practical and easy to use
- ✓ Equipped with disposable film pouch
- ✓ 专利设计
- ✓ 经济、实用，使用方便
- ✓ 配备一次性薄膜袋使用

Product Selection

Reorder No.	Description	Quantity
4046	50mm	40/10/Cs

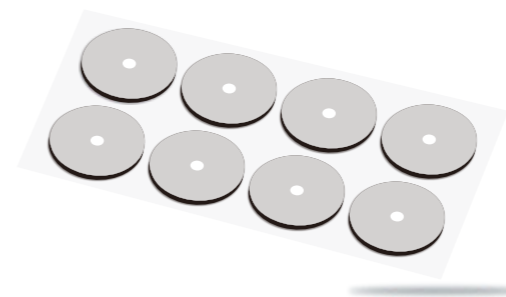


High-efficiency Activated Carbon Filter

活性炭高效过滤器

- ✓ Effective for odor removal
- ✓ Easy to use
- ✓ 过滤异味，效果好
- ✓ 使用方便

Product Selection

Reorder No.	Description	Quantity
4049	—	20/80/Cs

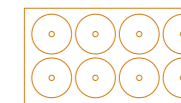


Skin Protective Ring 防漏圈

- ✓ Skin protective ring
- ✓ Flexible size
- ✓ 皮肤保护圈
- ✓ 尺寸灵活

Product Selection

Reorder No.	Description	Quantity
4047	Φ50mm	100/10/Cs

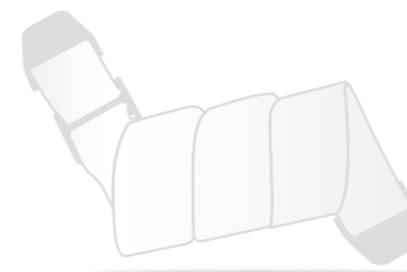


Ostomy Belt 造口腰带

- ✓ Ostomy Belt
- ✓ White
- ✓ 造口腰带
- ✓ 白色

Product Selection

Reorder No.	Description	Quantity
4050	2.9cmx10cm	100/1/Cs

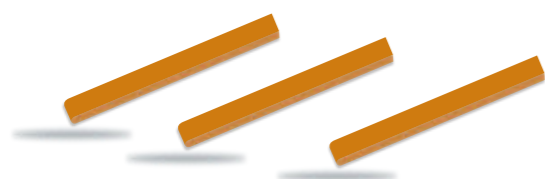

Anti-leakage Strip 防漏条

- ✓ Skin protective strip
- ✓ Flexible size
- ✓ 皮肤保护条
- ✓ 尺寸灵活

Product Selection

Reorder No.	Description	Quantity
4048	9mmx10.5cm	100/10/Cs




Skin Protective Powder 造口护肤粉

- ✓ Skin Protective Powder
- ✓ 护肤粉

Product Selection

Reorder No.	Description	Quantity
4051	1oz (28g)	100/1/Cs




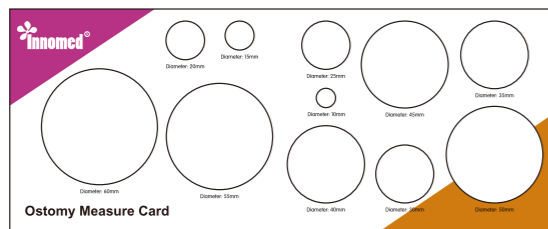

Ostomy Measure Card 造口测试尺

It is used for measuring the size of stoma so as to cut the hole of the skin barrier accordingly.
 用于测量造口大小，以便正确剪裁胶盘开口。

Product Selection		
Reorder No.	Description	Quantity
4052	—	—

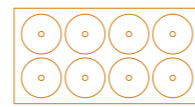

Urostomy Drain Tube Adapter 尿路引流连接管

Urostomy Drain Tube Adapter
 尿路引流连接管

Product Selection		
Reorder No.	Description	Quantity
4055	—	—



Skin Protective Paste 防漏膏

Skin Protective Paste
 防漏膏


Product Selection		
Reorder No.	Description	Quantity
4053	2 oz (60g)	100/1/Cs



Open Pouch Clip 开口袋夹

Open Pouch Clip
 开口袋夹


Product Selection		
Reorder No.	Description	Quantity
4056	—	—

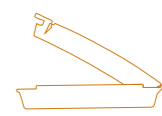

Skin Protective Spray 造口皮肤保护剂

Skin Protective Spray
 造口皮肤保护剂

Product Selection		
Reorder No.	Description	Quantity
4054	3cmx7cm	20/30/Cs

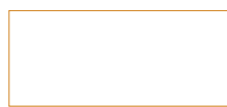

Aluminum Strip For Open Pouch 开口袋粘铝条

Aluminum Strip for Open Pouch
 开口袋粘铝条

Product Selection		
Reorder No.	Description	Quantity
4057	—	—

